


# attivo™

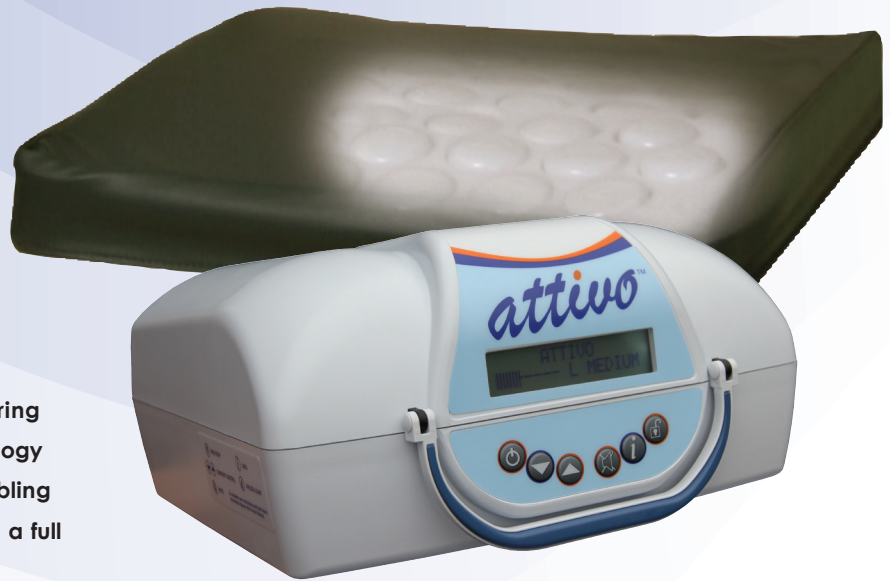

Suitable for the prevention and treatment of pressure ulcers within all care environments, the ATTIVO™ seating systems provide pressure relief and reduced interface pressures for patients at high risk of pressure damage. The seating systems feature an alternating air pressure cycle to provide a continuous pressure-relieving wave, to help to promote circulation to areas normally occluded during prolonged seated periods. Dual power technology offers both mains and battery operation, enabling at least 24 hours of continuous running time on a full battery charge.




### Simple and Efficient Dual Power Pump

The versatile ATTIVO™ power unit benefits from dual-power technology, offering a seamless choice of mains or battery operation. The integral battery will operate the power unit continuously for at least 24 hours when fully charged. This enables the cushion to function for long periods away from a mains power supply, making it fully mobile for use with a wheelchair, as well as providing long-lasting power back-up, if mains power is unavailable.



**Dynamic and Adaptive Cushions** The ATTIVO™ seating systems comprises the ATTIVO™ power unit and either the B.A.S.E.™ SEQUENTIAL cushion or the B.A.S.E.™ RECLINER MAT, which provides a therapeutic support surface for the back as well as the seat. The cushions feature a dynamic, adaptive surface which provides pressure relief and comfort and can be used equally successfully in wheelchairs or armchairs.

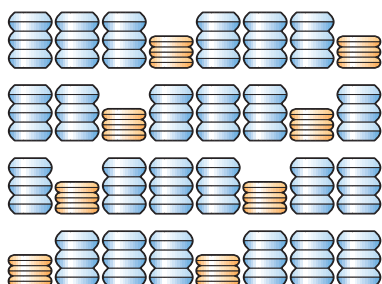

### Reactive Surface Combined with Active 1-in-4 Sequential Pressure Relief

The B.A.S.E.™ SEQUENTIAL cushion and B.A.S.E.™ RECLINER MAT feature strips of interconnected air-filled bellows, supported by a flame retardant foam surround. These bellows act as 'air springs', tilting and compressing, as the free movement of air between them allows each one to conform to the body's contours to provide reactive therapy, promoting a more even distribution of pressure. In addition, one row in every four is sequentially deflated to provide active pressure relief to areas at risk of pressure damage during prolonged seated periods.



## Clear and Simple

### Operation

Soft-touch operation buttons access the functions of the ATTIVO™ system. The power unit features audio alerts together with an LCD screen showing current operation status / fault conditions, e.g. comfort setting, battery status, low pressure, etc.



## Adjustable Comfort

### Control

Comfort levels can be adjusted to soft, medium or firm and are displayed on the LCD screen.



## Auto-Lock Feature

The system incorporates an auto-lock device to prevent accidental modification of settings once the seating system is in use.

## TECHNICAL DATA

**Power Unit Dimensions:** 210mm x 205mm x 105mm

**Power Unit Weight:** 2.2 kg

**Mode of Operation:** Active 1-in-4 cell cycle

**Alternating Cycle Time:** 16 mins

**B.A.S.E. SEQUENTIAL Cushion Dimensions:** 430mm x 430mm x 70mm

**B.A.S.E. RECLINER MAT Dimensions:** 1080mm x 430mm x 70mm

**B.A.S.E. SEQUENTIAL Cushion Weight:** 1.9 kg

**B.A.S.E. RECLINER MAT Weight:** 3.9 kg

**Cushion Inner Construction Materials:** PVC bellows within punched CMHR foam surround

**Cushion Cover Material:** PU coated stretch nylon

**Max. user weight guide:** 127kg / 20 stone

Talley manufacture products to comply with National and International safety standards and are certified to ISO9001, ISO13485 and Directive 93/42/EEC. Every care has been taken to ensure that the information contained in this brochure was correct at the time of going to press. However, Talley reserve the right to modify the specification of any product without prior notice in line with a policy of continual product development. Our standard terms and conditions apply. © Talley Group Limited 2016. All rights reserved.



**Talley Group Limited**  
Premier Way, Abbey Park Industrial Estate  
Romsey, Hampshire, SO51 9DQ England  
TEL: +44(0)1794 503500  
FAX: +44(0)1794 503555  
EMAIL: sales@talleygroup.com



[www.talleygroup.com](http://www.talleygroup.com)



**Ideal for all care environments**



**For patients at high risk of pressure ulcers**



**Dual power technology for mains or battery operation**



**PHASE FLOW™ energy-efficiency**



**Active 1-in-4 cycle**



**Adjustable comfort control**



**Auto-lock device prevents accidental modification of settings**



**Audible and visual system status/fault indicators**



**Vapour-permeable, waterproof, two way stretch cover**



**Welded cover seams**



**Modular parts used throughout so simple to repair/service on site**



**Supplied with versatile carry bag**



**2 year manufacturer's guarantee (power unit)**



**Please ask about daily rental**